

PUTTING ON SILVER WHINNYS®



-1-



-2-

1. If your horse will allow, put a gallon plastic bag over the hoof.
2. Slide the sock over the bag, completely over the hoof and onto the leg.
3. Remove the bag and put the hoof down on the ground.
4. Adjust the sock up and down on the leg. The colored yarn indicates the bottom of the sock. Do not twist the socks. Keep the ribbing straight.

Use duct tape to cover the rough spots on the hoof if your horse won't tolerate a plastic bag.

ONLY USE TWO SOCKS FOR THE FIRST 24-48 HOURS. GIVE YOUR HORSE TIME TO GET USE TO THEM! IF THERE ARE ANY ISSUES, THEY WILL APPEAR IN THE FIRST 24-48 HOURS. DO NOT ADD THE SECOND PAIR IF THE FIRST PAIR ARE NOT PERFORMING CORRECTLY.

WE HAVE GREAT TECH SUPPORT. CALL US AT 850-907-5724 OR EMAIL AT: SOXFORHORSES@GMAIL.COM



-3-



-4-



-5-



-6-

THE INSTRUCTIONS ON THE BACK ARE YOUR RECIPE FOR SUCCESS. TAKE TIME TO READ THEM SO THAT YOU AND YOUR HORSE WILL ENJOY THE BENEFITS OF THE SOCKS.

DO NOT ride or exercise your horse while it is wearing Silver Whinnys without the use of a sport boot to help the socks stay in place. Bell boots will not work. Check them often until you are sure the sport boot is holding them in place.

Sox For Horses, Inc. disclaims any and all liability for damages, injuries or death To persons, property or animals resulting from failure to follow instructions, or use Silver Whinnys® properly, or for injuries resulting from efforts to place Silver Whinnys® on an equine.